

EASYPPOINT[®]

retractable needle

- AUTOMATED NEEDLE RETRACTION
- EASY TO USE - REQUIRES MINIMAL TRAINING
- MAY BE ACTIVATED BEFORE OR AFTER REMOVAL FROM PATIENT
- PRE-REMOVAL ACTIVATION REDUCES THE RISK OF EXPOSURE TO THE CONTAMINATED SHARP
- ACTIVATION CAN BE ACHIEVED IN A VARIETY OF WAYS
- EFFECTIVELY REDUCES THE RISK OF NEEDLESTICK INJURIES
- PREVENTS RECAPPING AFTER USE
- REMAINS SAFE THROUGH DISPOSAL

compatible with both luer lock & luer slip syringes (3 mL or smaller)



may be used with pre-filled syringes



EASYPPOINT[®]

RETRACTABLE NEEDLES

(50 per box/400 per case)

<i>Catalog #</i>	<i>Needle Size</i>
82091	25G x 5/8"
82011	25G x 1"
82031	23G x 1"

OTHER AVAILABLE SAFETY PRODUCTS

VanishPoint[®] Syringe

VanishPoint[®] Blood Collection Tube Holder

VanishPoint[®] Blood Collection Set

VanishPoint[®] IV Catheter

Patient Safe[®] Syringe

NEW **EASYPPOINT**[®]

retractable needle



— **EASY TO USE** —

- minimal change in injection technique
- allows for single-handed activation
- **clinician's hands and fingers remain behind the exposed sharp** at all times



1
Attach EasyPoint needle to any luer lock or luer slip syringe. Administer medication according to institutional policy.




2
Push the color-coded tab, allowing the needle to automatically retract into the safety chamber. **This device is designed for activation before or after removing the needle from the patient.**



3
Dispose in an appropriate sharps container per protocol of institution.

*3mL syringe or smaller

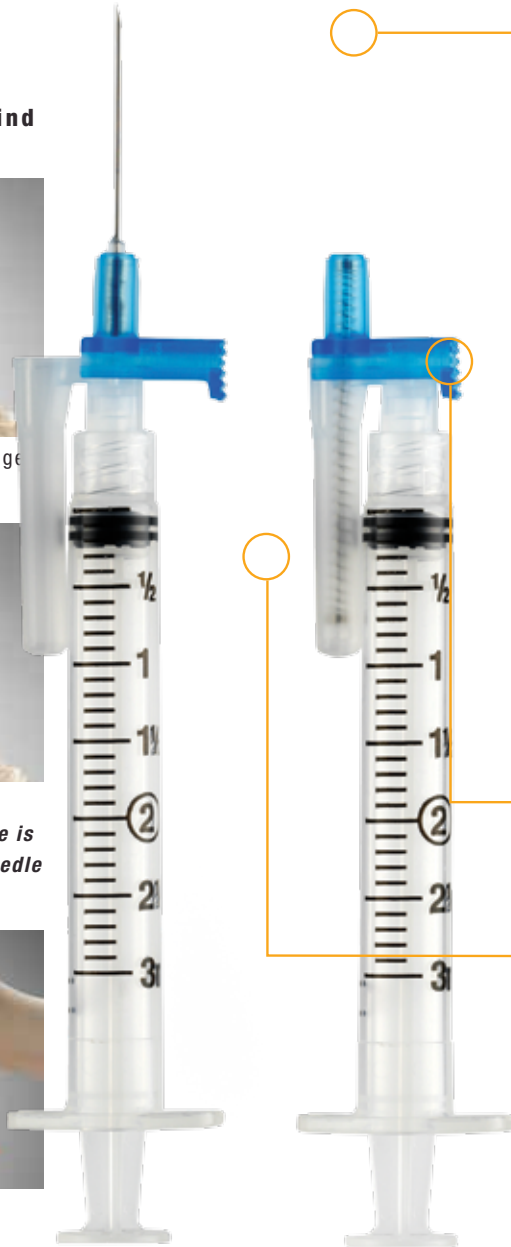

Triple beveled, lubricated needle provides patient comfort

WHEN ACTIVATED,
NEEDLE RETRACTS
AUTOMATICALLY

Standard luer fitting provides secure, leak-resistant connection

After retraction, contaminated needle is enclosed in safety chamber preventing exposure

Needle hub & activation tab are color-coded for needle gauge

Clear, compact safety chamber does not interfere with injection

*AUTOMATED RETRACTION
— now with —
ADDED VERSATILITY*

EASYPOINT[®]